DE

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## EN

Beppy condoms are made of high quality natural rubber latex, smooth surface, natural color, including reservoir and equipped with a non-medicated lubricant. If the manufacturer's instructions are followed, the condoms are an effective contraceptive method. However, no contraceptive method offers 100% protection. When used properly, condoms help reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases, including chlamydia, herpes, genital warts, gonorrhea, hepatitis B and syphilis.

Nominal width :  $56 \pm 2$  mm.

### Precautions and warnings:

- The condom is for single use only, otherwise there is an increased risk of failure or infection.
- Do not use oil-based lubricants such as Vaseline, massage oil, baby oil, body lotions, butter, etc. which may damage the condoms. Some medications applied to the penis or vagina can also damage condoms. Ask your pharmacist or general practitioner for advice.
- In case of irritation, stop using the latex condom and seek advice from your family doctor.
- Seek medical attention, at least within 72 hours, if the condom tears or leaks during use.
- If the foil is damaged, discard the condom.
- Store in a cool, dry place, out of the influence of direct sunlight.
- Stop and check if the condom slips or if the condom is too tight around the penis, as this may cause the condom to break.
- If the rubber material is sticky or brittle, do not use it.

### Instructions for use:

**STEP 1:** Before use, check the expiration date. Tear open the package on one side and carefully remove the condom. Condoms can be damaged by nails and jewelry.

**STEP 2:** Do not apply the condom until the penis is completely rigid. Avoid any contact between the penis and your partner's body before the condom is applied to prevent pregnancy and transmission of STDs. Air in an applied condom, can cause breakage. Avoid this by pinching the tip of the condom closed. Make sure the part to be unwound is on the outside and place the condom on the top of the erect penis. Continuing to hold the tip, use your other hand to unroll the condom along its entire length. If the condom is applied properly the reservoir did not contain an air bubble.

**STEP 3:** After ejaculation, hold the condom at the edge and pull the penis back even before it slackens. This helps prevent slippage.

**STEP 4:** After use, roll the condom in a piece of tissue and throw it in the trash. Do not flush down the toilet.

**WARNING:** This condom is made of a rubber latex which can cause allergic reactions including anaphylactic shock. This can happen when the user is allergic to latex.

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